

# Application Note 23

Testing with the bellavista Verification App on  
CITREX H5

analyser  
the art of measuring

**IMT Analytics released the CITREX H5, the latest testing device in the ventilation industry, in March of 2017. Its Android operating system set a new benchmark for usability and visualisation. The versatile user interface and its powerful processor support the user in testing different devices and analysing data.**


**The ability to run apps on the CITREX H5 simplifies the verification testing of medical devices. With the first App available, the Biomedical Engineer now has a powerful tool to test the bellavista ventilator.**

## bellavista ventilator

The bellavista ventilator family consists of three different ventilators, the bellavista 1000, bellavista 1000e and bellavista neo. All three fully featured ventilators were designed for the requirements of intensive care patients. In demanding and complex work situations the user is supported with powerful ventilator tools and an easy to use user interface. The combination of an internal battery, with up to six hours of operating time, and a turbine simplify safe in-house patient transportation. Like all mechanical ventilators, the bellavista series ventilators need preventive maintenance once a year. One part of that maintenance is the performance verification testing.

### Editorial

**Marco Kellenberger**  
Product Manager  
IMT Analytics AG

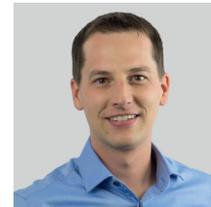


**IMT.Analytics**

IMT Analytics AG . Gewerbestrasse 8 . 9470 Buchs . Switzerland . [www.imtanalytics.com](http://www.imtanalytics.com)

## App overview


The “bellavista Verification App” fulfils the needs of every Biomedical Engineer working with our ventilators. IMT Analytics designed the app to support and simplify the complete verification process and to reduce human error. The testing can be performed quickly and accurately with the step-by-step process supported by text and images. At the end of the verification process a PDF report is generated and stored. The reports can be viewed directly on the device and exported via USB.

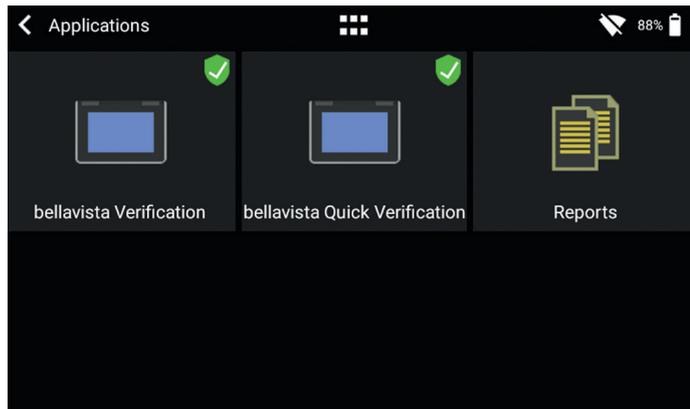

Overview of the apps and reports

## Preparation

Each time the app starts a zero offset calibration must be performed. This guarantees the best possible measurement results. An oxygen sensor calibration is optional and only has to be carried out if necessary. Before taking a measurement for a test step; the tubing setup, circuit test, and ventilator settings adjustments must be performed. All of these steps are supported by images and some of them can be enlarged by tapping on them.

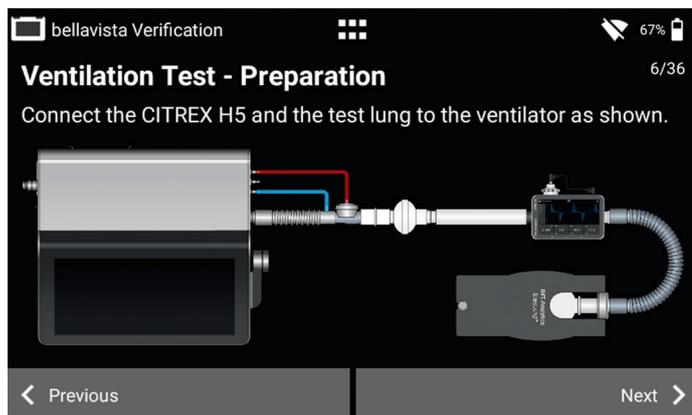

Setup the tubing for the measurement

## Test steps

After preparing the device and starting ventilation, the CITREX H5 is ready to measure. Automatic measurements are divided into four columns, "Parameter", "Measured", "Expected" and "Result". The actual value of each parameter is shown in the "Measured" column. The "Expected" values are the tolerance range according to the requirements of the service manual. If the measured value is within the range, the result shows a green checkmark. Otherwise, the result shows a red X. As soon as the values are stable, the user can press "Next" to store the actual value for the report and move to the next screen. It is possible to tap next if the measurement is out of range to continue testing.


Parameter	Measured	Expected	Result
P <sub>Peak</sub>	17.3 mbar	23 .. 28 mbar	✗
PEEP	5.0 mbar	4 .. 6 mbar	✓
Rate	14.0 b/min	14 .. 16 b/min	✓
O <sub>2</sub>	21.0 %	19 .. 23 %	✓

Some tests require a verification of parameters or action on the bellavista screen. In this case, the app displays check boxes which the user checks to confirm the action was performed. Tapping next stores the results for the test report and move to the next screen.

Parameter	Expected	Result
P <sub>Peak</sub>	23 .. 28 mbar	✓
PEEP	4 .. 6 mbar	✓
Rate	14 .. 16 b/min	<input type="checkbox"/>

## Advanced ventilation test

The advanced ventilation test is optional. If the switch next to the title is off all pages from this test step are skipped. The default setting is on off. The test consists of preparation, measurement and check boxes.

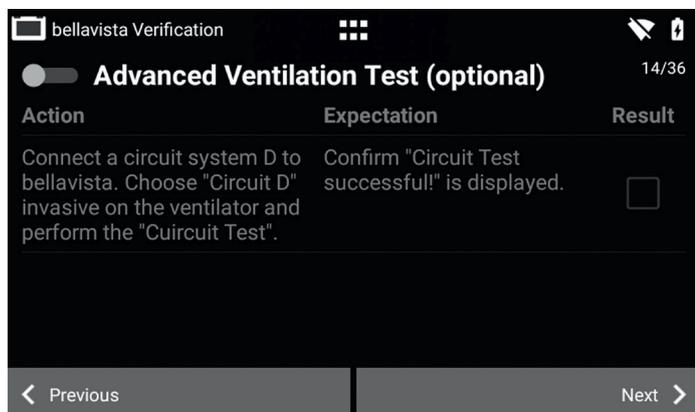


Advanced ventilation test with on/off switch

## General Information

The CITREX H5 collects various information. All the information is used to generate a test report. The information about the bellavista ventilator, such as serial number, operating hours, model and customer information, must be entered manually.

The information about the Test Center is prefilled if they are stored in the CITREX H5 settings. The company information and the Operator/Tester can be edited. The tester can sign the report on the multi-touch display. Tapping "Finalize" generates a PDF report which cannot be modified.

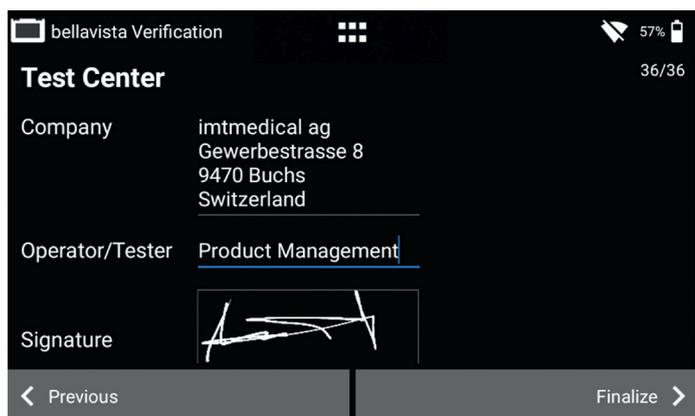

Information about the Test Center

# Test reports

All test reports generated with the CITREX H5 are stored on the device. Each receives a unique number in its file name. In the section “Reports” under “Applications” it is possible to view the reports on the screen of the CITREX H5. To export selected, or all, PDF test reports connect a USB thumb drive and choose the menu icon in the Reports section.

<b>IMT Analytics AG</b> Gewerbstrasse 8 9470 Buchs Switzerland				
<b>bellavista Verification - Report</b>				
<b>Test Equipment</b>	<i>Device</i>	IMT Analytics AG CITREX H5		
	<i>Serial Number</i>	BB10002		
	<i>Calibration Date</i>	02.03.2017		
<b>Test Object</b>	<i>Customer</i>	Hospital XYZ		
	<i>Department</i>	ICU		
	<i>Type</i>	imtmedical bellavista 1000e		
	<i>Serial Number</i>	MB102235		
	<i>Operating Hours</i>	12'352 h		
<b>Test Results</b>				
<b>Oxygen Supply Check</b> <span style="float: right;">✓</span>				
<b>Circuit C Test</b> <span style="float: right;">✓</span>				
<b>Ventilation Test</b>				
	<i>Parameter</i>	<i>Measured</i>	<i>Expected</i>	<i>Result</i>
	P <sub>peak</sub>	24.7 mbar	14 ... 20 mbar	✗
	PEEP	5.2 mbar	4 ... 6 mbar	✓
	Rate	15.2	11 ... 13	✓
	FiO <sub>2</sub>	21.2 %	19 ... 23 %	✓

Sample test report

« The Biomedical Engineer now has a powerful tool to test the bellavista ventilator »