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To perform the oxygen calibration, tap on × until «O₂ Calibration» is shown on the screen. Choose with the ○ button between a one-point calibration and two-point calibration and follow the instructions on the screen. When carrying out the two-point oxygen calibration, ensure you use 100% oxygen. For further information please refer to the user manual.

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Press ○ to choose mode
Press △ to start

User manual Bedienungsanleitung

The user manual for the CITREX H3 can be found on our website at the link below. Please read the operating instructions carefully and use the product as recommended.

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Read the user manual carefully before using the CITREX H3.
Online at www.imtanalytics.com/analyser_manuals

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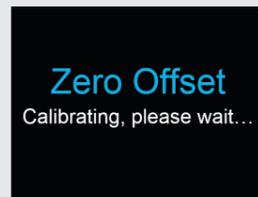
Calibration Kalibrierung

EasyCal

Worldwide express 10-day service including shipping. More information on:
www.imtanalytics.com/easycal

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To guarantee the accuracy of your measurements, your CITREX H3 must be recalibrated by IMT Analytics yearly.

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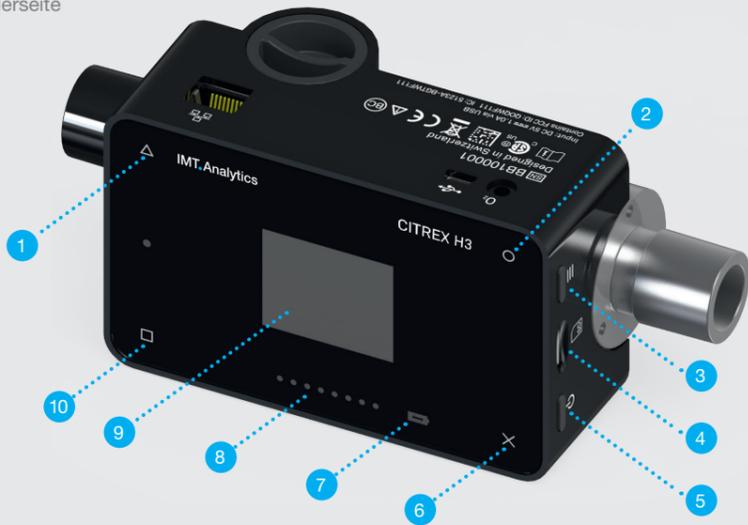
Setup for flow measurement

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It is recommended to use a filter for all measurements. The filter protects the device from dust and contamination. Please place the filter with a tube in front of the device.

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The measured gas must be free of oil, grease and dust.

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Negative Flow Direction
Negative Flussrichtung

The device setup can affect the accuracy of the flow measurement. Tight bends, kinks or dents should be avoided. For more information please consult the user manual.

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Bad setup: Kink, tees, angle pieces at the device inlet

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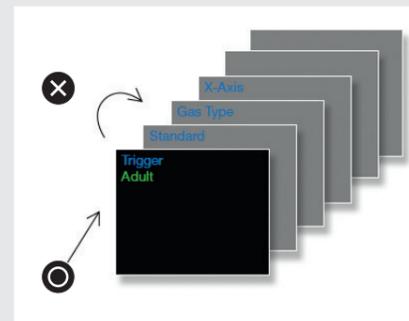
Change settings

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Tap the X-symbol to open the settings menu. By tapping this symbol you can scroll through the settings. Use the O-symbol to change the settings.

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
Numerical Values

Numerische Werte

Numeric values are displayed by tapping the □-symbol or on the Δ-symbol. By tapping on the symbol will scroll through the six configurable screens.

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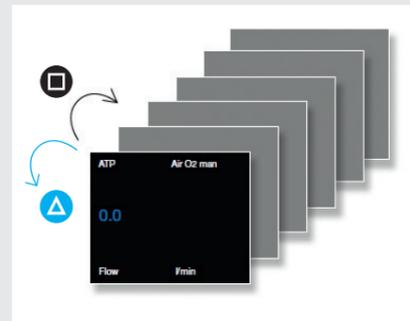
Change parameter and units

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If the context button (≡) is pressed twice in quick succession, the "Edit Mode" is activated. A red icon on the screen indicates this. The parameter or the unit in the red frame can be changed with the Δ-symbol or the □-symbol. The O-symbol has the function to jump to the next parameter or unit. If the context button or the X-button is pressed once, the "Edit Mode" is exited.

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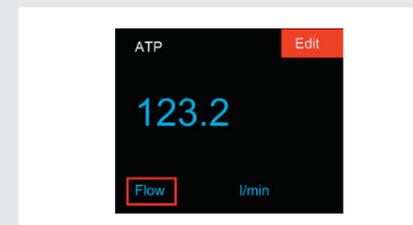Measurement Setup for Mechanical Ventilators

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The CITREX H3 is a powerful device for ventilator calibration and verification. We recommend you follow the measurement setup below.

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IMT Analytics AG . Gewerbestrasse 8 . 9470 Buchs . Switzerland